DD Nature Zr

自然锆饰面瓷粉 | Ceramic Bonding

使用说明 | *Instructions for use*

完美品质, 触手可及 | Quality, available for *everyone*

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If you expect more than **standard**, but want a fast and **secure** way to reach your goal.

We live up to our own demands for brilliant, aesthetic dental prostheses with the innovative **DD Nature Zr** porcelain veneers.

The name is a commitment – almost identical to the natural model.

The concept allows you to achieve lively, reliable colour renditions with the least effort. A veneer system that adapts to your laboratory routine and not vice-versa.

As a ceramic specialist and one of the leading zirconia producers, we ensure that all processes are coordinated with each other, from the raw material to the finished milling blank and ceramic powder. We provide you with the basis for strong, flawless bonding practically ex works.

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DD Nature Zr is a veneering ceramic that is optically and technically perfectly attuned to zirconia frameworks (Y-TZP). TEC has been optimised for very strong bonding with zirconia (TEC $10.5 \cdot 10^{-6} K^{-1}$).

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The best results can be achieved when veneering DD Bio Z frameworks.

DD Nature Zr is unsuitable for veneering metal or titanium frameworks

DD Nature Zr is solely intended for dental use.

储存在干燥、避光的环境中
Store in a dry place and protected from light.

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You can create the ideal basis for individual veneering with **DD Bio Z color liquid**. Thanks to the perfect interplay of the coloured framework and veneering ceramic, shoulder ceramics or liners can be dispensed with.

Depending on the zirconium framework material, following Liquids are available:

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贴面
frames and coppings

16 x 牙本色 | *dentine shades*:
浸泡
dipping
A1 - D4

请注意，每种染色剂的详细的操作说明。
Please note the detailed working instruction for each liquid.

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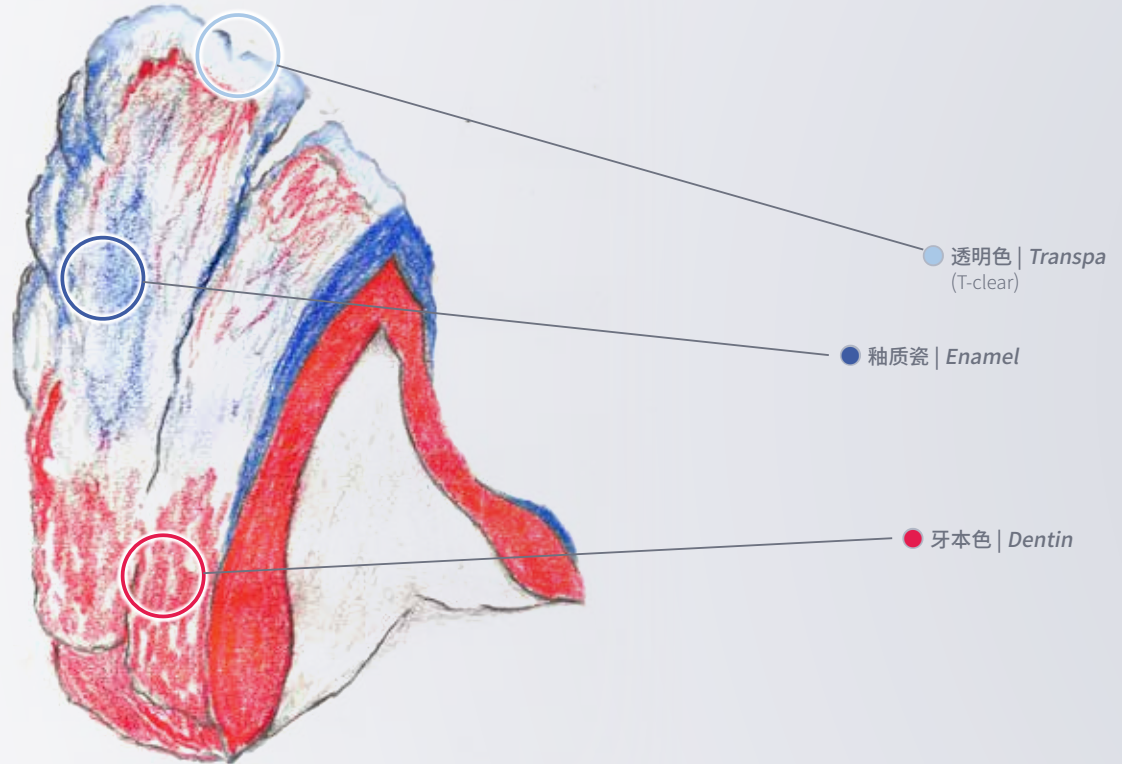
The firing parameters given above are guidelines, which must always be adjusted to suit the furnace used and the situation of the furnace. What is essential is getting the right firing result. Best way is to fire a test body made of T-clear and compare the transparency of the supplied indicator.

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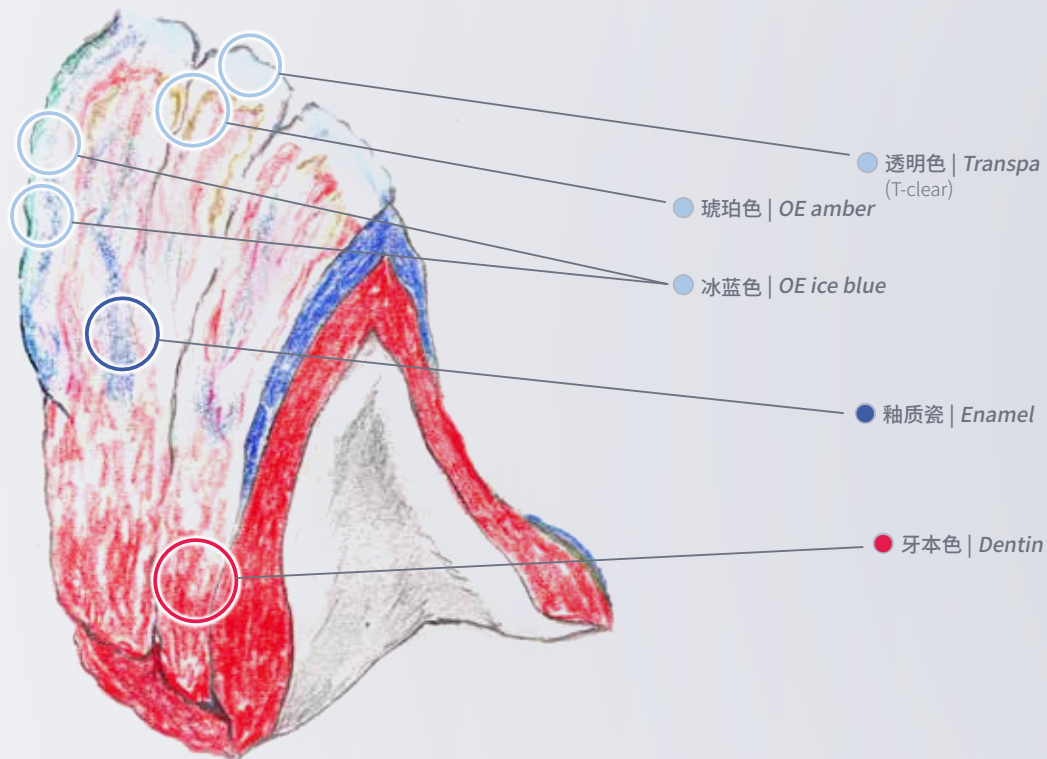
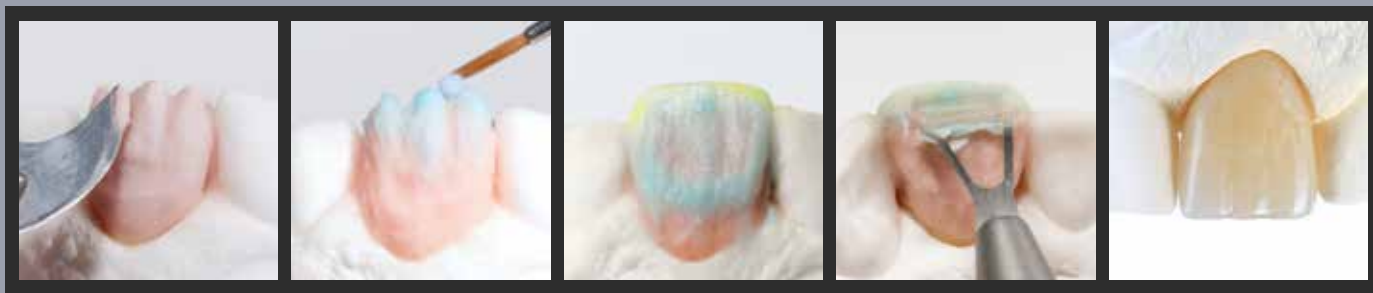
The labial and occlusal form is first built up with dentine material in the required shade. Once the anatomical form is ready, reduce the incisal and interdental areas of the dentine in order to create enough space for further layering.

Then complete the missing areas with enamel and transparent porcelains or effect transpa / transpa opal porcelains – depending on the individual appearance of the crown required.

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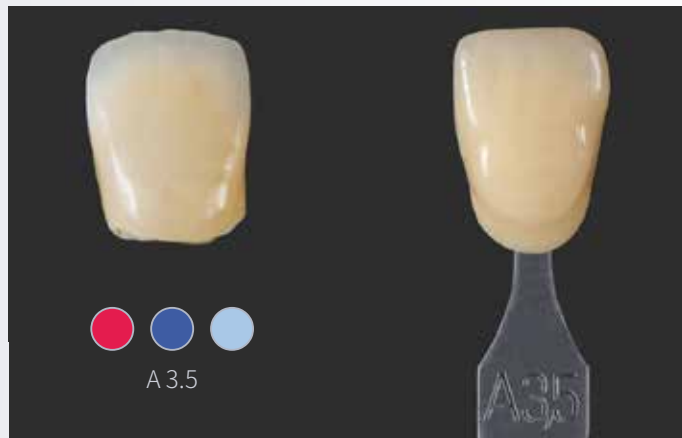
The correction firing serves to optimise the form and supplement the fire shrinkage.

Before layering, the crown must be prepared and cleaned.

The glaze firing occurs after preparation, surface contouring and of course final cleaning.

For DD Nature Zr and full anatomical constructions made of DD Bio Zr translucent the paste paints from Dental Direkt provide the perfect finish.

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Tip:

Thanks to the perfect dentin body colouring using DD Bio Z color Tclassic liquid, economical single layering (enamel) is also possible as an alternative.

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The DD Zirconium Academy

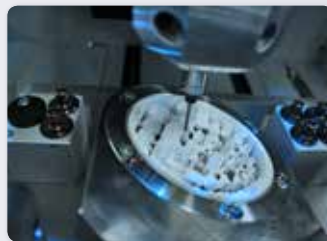
– Perfect combination of technology and material.

In our application laboratory all indications can be manufactured under guidance from the simple and efficient production of a DD Bio ZX² monolith-crown or the entire range of our DD Nature Zr veneering porcelain.

*With our **DD TEC**-systems we impart*

to you the techniques of basic CAD/CAM-knowledge up to the production of complex, full-ceramic implants.

Our trainers work practically and for every-day life. The goal is to deal generally with your individual requirements and optimise working processes.

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**Dental Direkt GmbH**